


PDM Menu Map

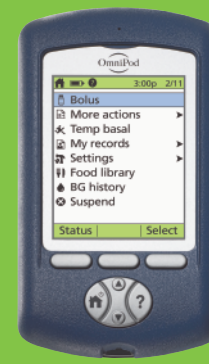


Insulin Management System

THE PDM IS EASY TO NAVIGATE—BUT SOMETIMES IT IS JUST NICE TO KNOW WHAT THE OPTIONS ARE. THIS MAP SHOWS PRECISELY WHERE EVERY FUNCTION IS LOCATED.


HOME SCREEN

Bolus

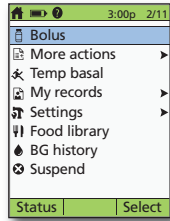


More actions >

- Change Pod
- Add BG reading
- Assign/Edit BG tags

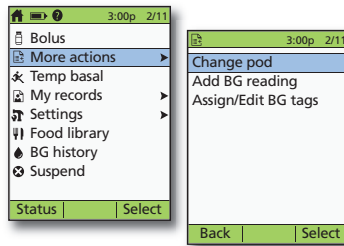


Temp basal

(will not display on menu if Temp basal = Off in System setup)

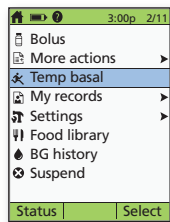


My records >

- Insulin delivery
- BG history
- Alarm history
- Carb history
- All history
- My info

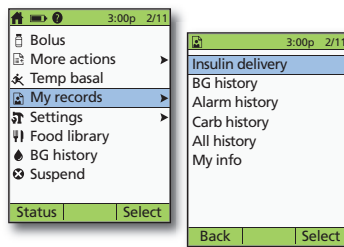


Settings >


- Basal programs
- Presets >
 - Temp basal presets
 - Carb presets
 - Bolus presets
- System setup > (see right panel)

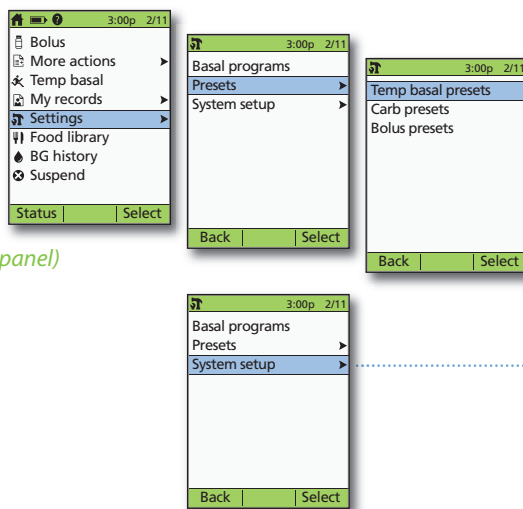

Food library

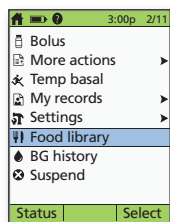


BG history

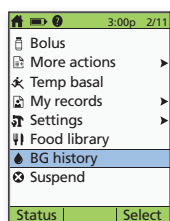

Suspend


(is replaced with "Resume" when insulin is suspended or "Suspend/cancel" when a temp basal or extended bolus program is running)

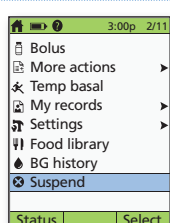

System setup >


• Date/time >

Time
Date: MM/DD/YY

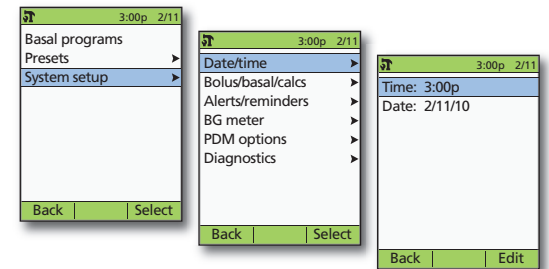


• Bolus/basal/calcs >

Bolus calcs: **On**
Ratios/factors/targets > (will not display on menu if Bolus calcs = Off)
Review all settings
Target BG: ---
Min BG for calcs: **70 mg/dL**
IC ratio: ---
Correction factor: ---
Reverse correction: **On**
Insulin action: ---
Temp basal: **Off**
Extended: **Off**
Bolus increment: **0.10 U**
Max bolus: **10.00 U**
Max basal: **3.00 U/hr**

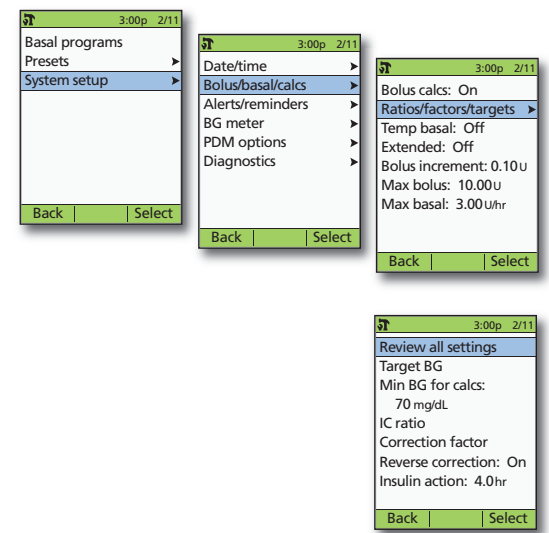


• Alerts/reminders >

BG reminder: **Off**
Expiration: **4 hr**
Low reservoir: **10.00 U**
Auto-off: **Off**
Bolus reminders: **Off**
Reminder alerts: **On**
Confidence alerts: **On**
Custom alerts

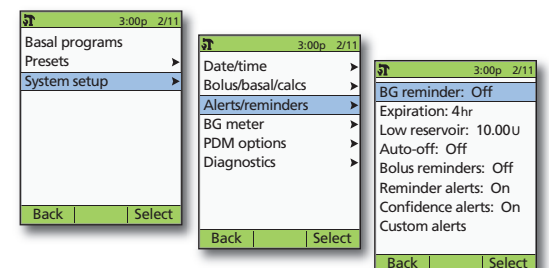


• BG meter >

BG goal: **100 - 100 mg/dL**
Manage BG tag list >
Standard BG tags
Custom BG tags
BG sound: **On**

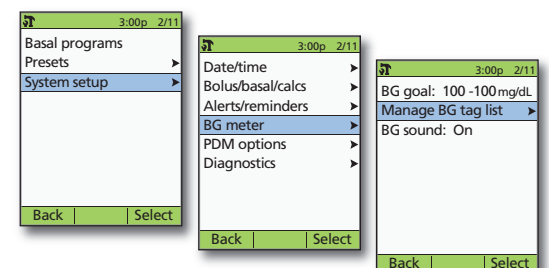


• PDM options >

PDM lock: **Off**
Screen time-out: **20 sec**
Backlight time-out: **10 sec**

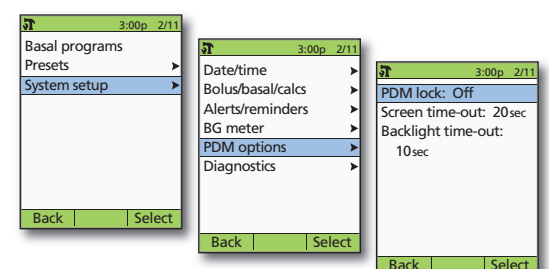

• Diagnostics >

Check alarms
Reset PDM

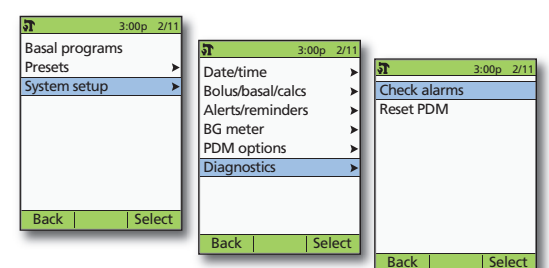

NOTE: PDM entries in blue indicate default values. All values shown are for illustrative purposes only. Consult with your healthcare provider to determine the appropriate settings for you.

This guide is for PDM model UST200. The PDM model number is written on the back cover of each PDM.

PDM Menu Map



Insulet Corporation
9 Oak Park Drive
Bedford, MA 01730 USA
800.591.3455 / 781.457.5098
MyOmniPod.com



Making diabetes a smaller part of life.