


# Placement Options

*No strings attached.™*


- Because there is **NO TUBING**, you can wear the Pod comfortably anywhere you would give yourself a shot.

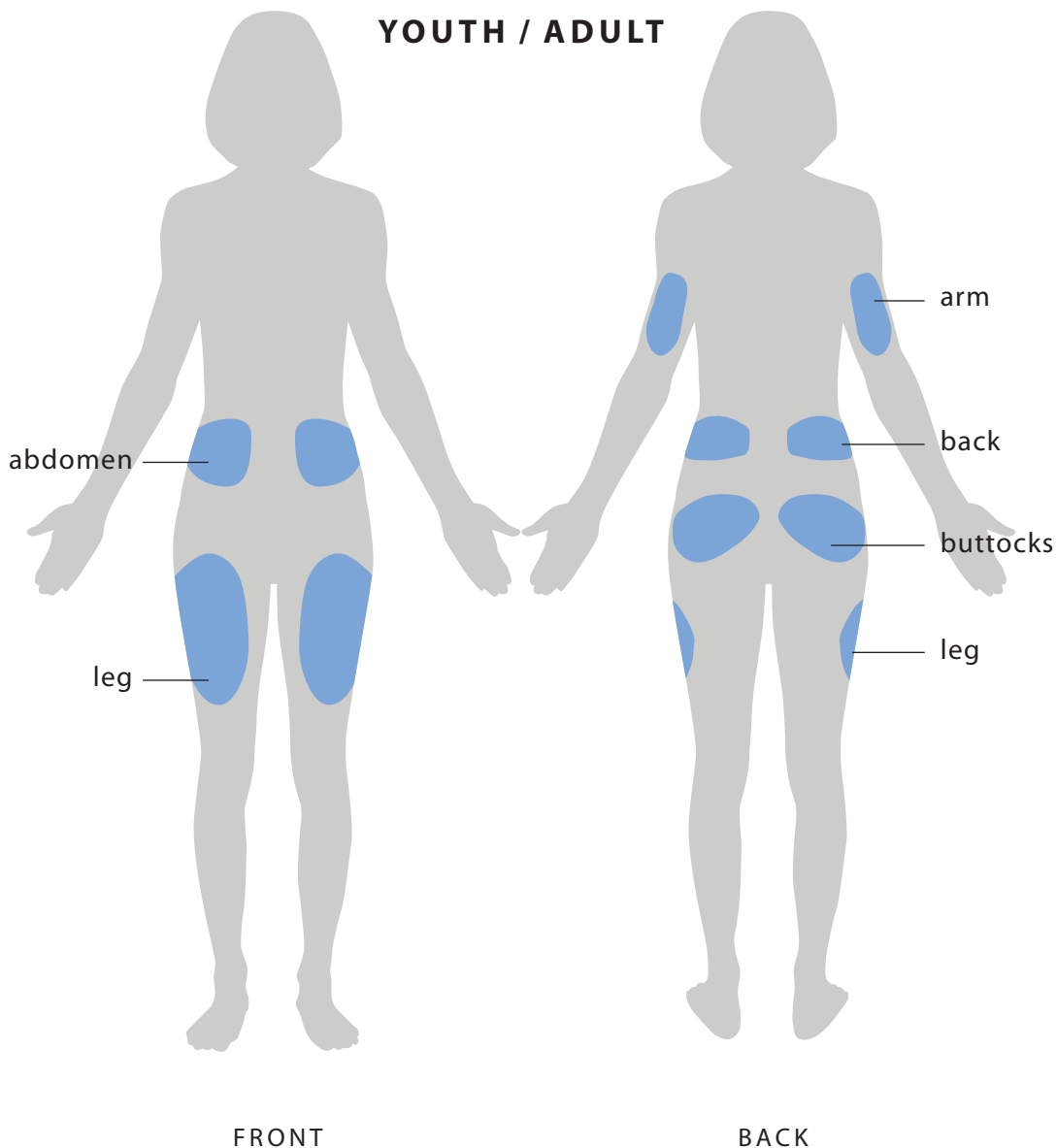

## PLACEMENT TECHNIQUES FOR BEST RESULTS

### Pod Positioning

**Arm and Leg:** Position the Pod *vertically* or at a slight angle.

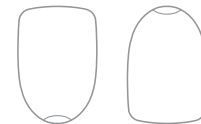

### Abdomen, Back and Buttocks:

Position the Pod *horizontally* or at a slight angle.



### Optimal Adhesion

Use an alcohol prep swab to clean the area.

Let site air dry thoroughly before applying Pod.

Do not place Pod where it will be affected by folds of skin.

### Pinching Up

Place your hand over the Pod, pinch up the skin near the cannula, then press the Start button on the PDM. Release when cannula inserts.

This step is critical if the insertion site is very lean or does not have much fatty tissue.

# Placement Options


*No strings attached.™*

- Because there is **NO TUBING**, a child can wear the Pod comfortably anywhere you would give them a shot.

## PLACEMENT TECHNIQUES FOR BEST RESULTS

### Pod Positioning

**Arm and Leg:** Position the Pod *vertically* or at a slight angle.

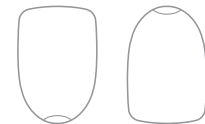

### Buttocks:

Position the Pod *horizontally* or at a slight angle.



### Optimal Adhesion

Use an alcohol prep swab to clean the area.

Let site air dry thoroughly before applying Pod.


Do not place Pod where it will be affected by folds of skin.

### Pinching Up

Place your hand over the Pod, pinch up the skin near the cannula, then press the Start button on the PDM. Release when cannula inserts.

This step is critical if the insertion site is very lean or does not have much fatty tissue.

## TODDLER

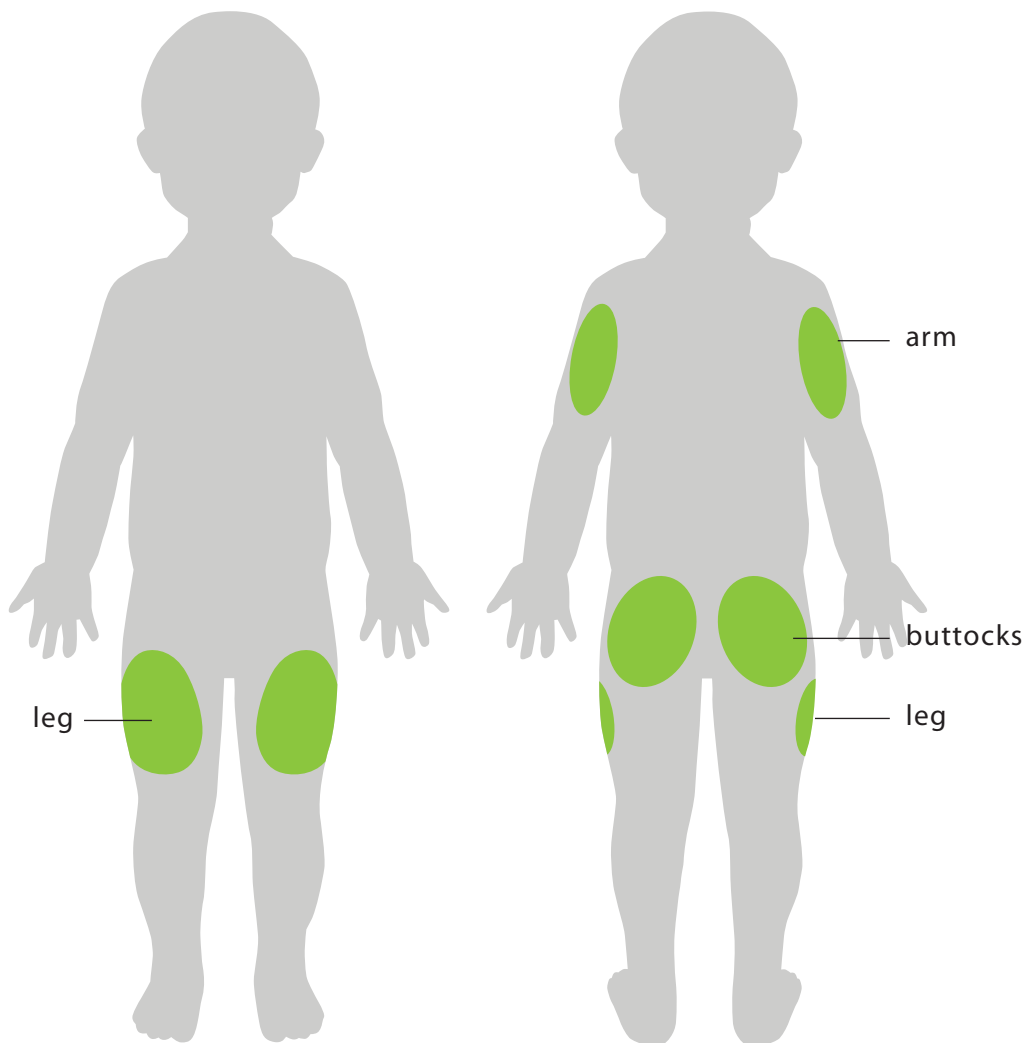

FRONT

BACK