


INSULET PROVIDED GLOOKO

# Create, Store and Print a PDF Report

Log into Glooko account on [my.glooko.com](https://my.glooko.com)

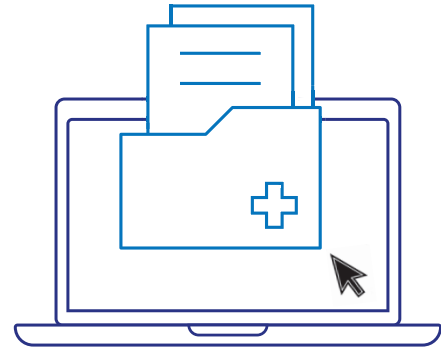


## Create a Report:

- Click patient's name
- Click **Create PDF Report** (top right)
- Choose report set and click **Create PDF**

## Store a Report:

- Save PDF report to computer
- Attach to patient's chart or other desired location
- Review in patient's chart, or print if desired

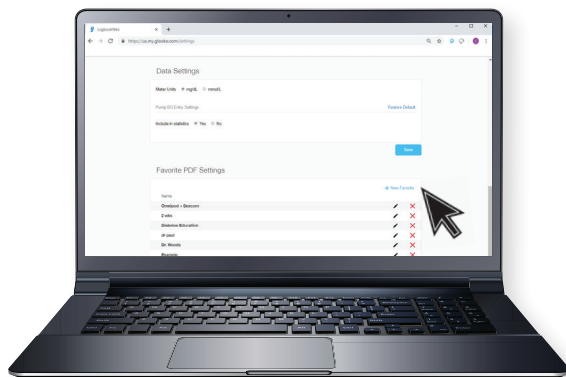

## TIP! Save Customized Report Sets:

- Click **Settings**
- Scroll down and click **New Favorite**
- Select desired reports, and name the report set (i.e. **Omnipod+Dexcom reports** or **Dr. Smith reports**)



For more information on the Omnipod® System,  
contact the Insulet Customer Care Team:  
**800.591.3455 | myomnipod.com**

---

If you are experiencing any issues with Glooko,  
please contact Glooko Support:  
(Call) **800.206.5601** | (Text) **650.720.5310**  
(Email) **support@glooko.com**